

AED CHECK

The Automated External Defibrillator (AED) is very simple to maintain and contains no user-serviceable parts. The AED performs a self-test every day. Periodic checks are required monthly which is limited to the following:

1. Check the green Ready light. This light must be blinking green.
2. Check for any used, damaged, missing or expired supplies and accessories. Be sure to check the expiration date on the Pads cartridge.
3. Check the outside of the AED for any cracks or other signs of damage.

If the AED emits a chirp this may be indication of a low battery.

Please initial and date once a month that the three items were checked and notify Security of any discrepancies.

AED Location: _____

	INITIALS	DATE		REMARKS
January:				
February:				
March:				
April:				
May:				
June:				
July:				
August:				
September:				
October:				
November:				
December:				

Case Contents:

- 1 – Defibrillator
- 1 – Carrying Case
- 1 – Battery
- 1 – pocket CPR mask
- 1 – Adult Pads Cartridge
- 1 – disposable razor
- 1 – Owner’s Manual
- 2 pairs – gloves
- 1 – Quick Reference Guide
- 1 pair – paramedic scissors
- 1 – Quick Start poster
- 1 – absorbent wipe

PLEASE NOTIFY SECURITY OF ANY DISCREPANCIES: SECURITY@NORTHWESTU.EDU.